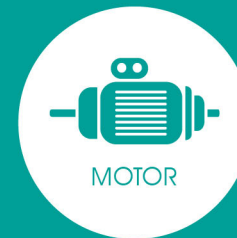
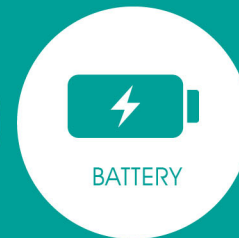
# HAPPY RIDER

*A power add-on system for upgrading  
a manual wheelchair*

MOTOR

BATTERY

CONTROLLER

## ALL-IN-ONE DESIGN

Main electronic parts are integrated inside the wheel

**DK**  **CITY**

- **Simple 3 steps, the ordinary manual wheelchair will be converted to an electric one**

- STEP 1. Fix the mounting bracket to the frame, swap wheels and fit with ball lock pin.  
 STEP 2. Attach joystick.  
 STEP 3. Connect the magnetic connectors between joystick and wheels.  
 Turn on the power, and now you are ready to go!

- **Switchable operating way**

Manual/electric can be switched at any time.

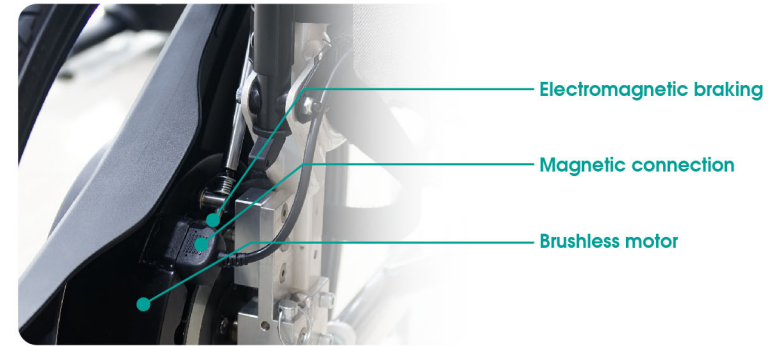
- **Intelligent operate**

With intelligent Joystick to handle straight and turning by one hand.  
 Anyone can operate easily.

Electric

Manual

- **Electromagnetic braking function**

Safety can be easily controlled even on slopes.

- **Magnetic connection**

The cable can be quickly separated from the magnetic head without the risk of being pulled break when remove wheel, making the process becomes easier, faster and more elegant.

- **Brushless motor**

Low defect rate and few maintenance.

SPEC	Power add-on system
Model Name	Happy Rider
Overall Weight	18.2 KG
*Wheel weight (Including motor, battery and controller)	8.5 KG / 1 wheel
*Joystick weight (with cable)	0.5 KG
*Mounting bracket weight	0.7 KG
Wheel Size	24"
Max User Weight	120 KG
Range	15 KM
Speed	0-6km/h
Max slope	3°(Manual mode) / 6°(Electric mode)
Motor	Brushless DC 24V 200W *2
Braking system	Electromagnetic brake 24V 10W *2PCS
Control system	Joystick Module
Battery	24V 5.8AH Li-ion Battery *2
Charger	INPUT :100~240VAC OUTPUT:29.2VDC 2A
Charging Time	7H

\* Range figure is based on the test result in laboratory. Actual figures may differ by using conditions such as user's weight, road conditions, and weather etc.

\*\* All specifications and product configurations are subject to change without notification.

<http://www.dkcity.com> E-mail: [sales@dkcity.com](mailto:sales@dkcity.com)

(P)+886-4-25321000 (F)+886-4-25325000

No. 1, Lane 160, Sec. 2, Tan-Fu Road, Tan-Tzu District, Taichung City, Taiwan R.O.C.

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